



Three-step application

1 Open device and place the carpoule inside it.



2 Close the device and screw in the cap with the syringe.



3 Select the dose*, remove the cap from the syringe and inject the medication.

*If you past the correct dose, screw to the end and select the dose again.



- ▣ Antirolling tabs.
- Multiple doses resetting system.
- Doses register scale.
- For multiple doses with pre-filled carpoule containing the medicine.
- Easy to transport. Personal use.
- Ergonomic size for convenient and accurate manual usage.
- Top-quality polymers that are highly resistant to falls and impacts.
- User's manual.